

♣ Cardio Quip MCH-1000(m) Quick Start Guide

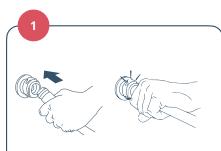

Attach Hoses

Connect the dripless connections to the corresponding "IN" and "OUT" connections in the back of the MCH.

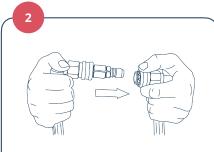

Attach Loopback Connector

Attach the end of the MCH water lines with a loopback connector. Make sure any inline valves are open.



Add Water & Ice

Fill the cold water tank with approximately 4L (1gal) of sterile or 0.22-micron filtered water and fill the remainder of the tank with ice made from sterile or 0.22micron filtered water.




Insert Power Connector

With the front panel mains power switch turned off, plug the power connector into a dedicated "hospital grade" AC power outlet. The outlet should match the voltage, frequency and capacity listed on the MCH label.



Power On

Switch on the power supply and ensure that the unit runs through a successful system test.

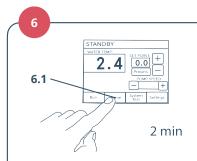


Prime

Press the **Prime** button (6.1) to clear the air from the water circuit. Allow the prime cycle to run for 2 min. (120 s).



Set Temperature: Three Ways

- 1. Press the SET POINT display (7.1) to manually input the water
- 2. Press the + or buttons (7.2) to adjust by 0.1°C.
- **3.** Press the Presets button (**7.3**) to set the temperature to a preset value.

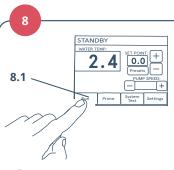

Press the Run button (8.1) to start the flow of temperaturecontrolled water